



An Ingersoll Rand Business



Sentinel XT HORIZON

Expanding Your Field of View

ILC Dover first introduced the Sentinel Clear Hood in 2010 as part of our Sentinel XL line of PAPRs, setting a new standard in respiratory protection. We continued to innovate with the Clear Hood and Clear Plus Hood for Sentinel XT™. Now, Sentinel XT™ Horizon builds on this legacy, offering enhanced protection and even greater ease of use.

Did you know you can trade your old PAPR for a new one with ILC Dover's trade-in program? We're here to help you stay ahead. [Call 302-546-5335 to learn more.](tel:302-546-5335)



Panoramic View

Enhances user safety compared to conventional hoods



Exceptional Protection

SWPF study average value exceeds 50,000 and APF rating of 1,000



Easy to Don

No adjustments required, simplifying the process



Universal Size


Fits a wide range of users comfortably



Secure Suit Attachment

Ensures a reliable connection to the suit

NEW Comfort Collar

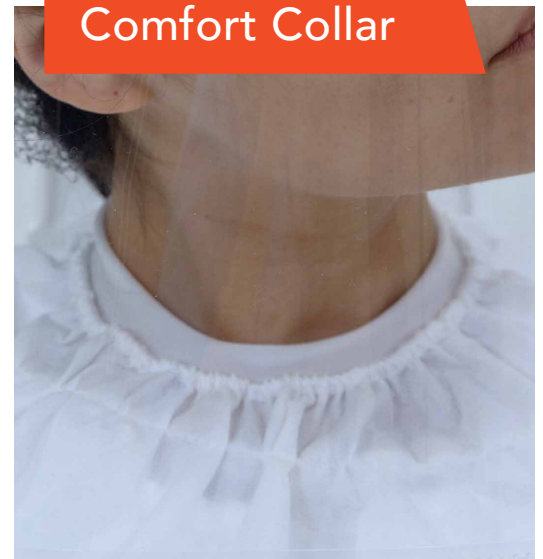

The universal breathable comfort collar delivers exceptional protection by maintaining consistent hood overpressure, making it optional to tuck the inner bib into the suit.

Quick & Easy Donning



1. Open the breathable collar.



2. Slide the hood over your head.



3. Release your hands and allow the hood to settle into place. Tuck in the inner bib if applicable and secure the outer bib clips to the suit.

Quick & Easy Doffing



1. Remove the bib clips and open the breathable collar.



2. Lift the hood off of your head.



3. Detach the hood from the breathing tube.

Available in Packs of 30

Sentinel XT
Horizon Hood



For more details on the
Sentinel XT™ Horizon,
visit our dedicated webpage.



An Ingersoll Rand Business

One Moonwalker Road
Frederica, DE 19946 USA
+1.302.335.3911
+1.800.631.9567
customer_service@ilcdover.com

www.ilcdover.com



See user manual for cautions and warnings. © Copyright 2024 ILC Dover LP